

Is your PiezoWave2 performing properly?



Step One: Check to see how many pulses are currently on your therapy source.

1. Press 3 and 5 simultaneously.

Step Two: Check the power of your therapy source.

1. Using the 0 mm Gel Pad, place your index finger on the tip of the Gel Pad.
2. Set the pulse frequency at 5.
3. Begin increasing the output intensity and note when the intensity becomes noticeable.



If the therapy source has over 5 million pulses or if the compression during the finger test is not felt until the machine output intensity is above 8, the therapy source is considered out of specification and should be replaced.

For further service questions contact Eddie Jentz
eddie.jentz@elvationusa.com | 770-295-0049 x231

