

Esteem Quick Reference Guide to your Personal Programmer

Button Quick Reference



Standby



Profile



Volume Up



Volume Down



Transmit



Query

Selecting a Profile or Turning ON your Esteem®

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Select your desired profile by pressing the “Profile” button.
3. Hold your Personal Programmer over your Esteem® and push the “Transmit” button. *

Changing the Volume

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Select your desired volume by pressing either the volume up or down button.
3. Hold your Personal Programmer over your Esteem® and push the “Transmit” button. *

Querying Your Current Esteem® Settings (e.g. after changing the programmer’s AAA batteries)

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Press the “Query” button.
3. With the “Query” LED illuminated, hold your Personal Programmer over your Esteem® and push the “Transmit” button.
4. Check Profile LED and Volume Setting Display. If the Profile LED and Volume setting are displayed, the Esteem® Query was successful. *
5. If the corresponding LED is blinking, communication was not successful; repeat previous steps. *

Envoy Medical Corporation
4875 White Bear Parkway
White Bear Lake, Minnesota 55110
United States of America
877-900-EARS (3277)
www.envoymedical.com

© Envoy Medical Corporation or its affiliates.

All rights reserved.

Esteem and Envoy Medical are trademarks of Envoy Medical Corporation.

006203-001 Rev 03

ENVOY MEDICAL