

# Esteem Quick Reference Guide to your Personal Programmer

## Button Quick Reference



### Selecting a Profile or Turning ON your Esteem®

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Select your desired profile by pressing the “Profile” button.
3. Hold your Personal Programmer over your Esteem® and push the “Transmit” button. \*

### Changing the Volume

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Select your desired volume by pressing either the volume up or down button.
3. Hold your Personal Programmer over your Esteem® and push the “Transmit” button. \*

\*If the corresponding LED is blinking, communication was not successful; repeat previous steps

### Querying Your Current Esteem® Settings (e.g. after changing the programmer’s AAA batteries)

1. Turn on your Personal Programmer by pressing the “Standby” button until your Personal Programmer illuminates.
2. Press the “Query” button.
3. With the “Query” LED illuminated, hold your Personal Programmer over your Esteem® and push the “Transmit” button.
4. Check Prole LED and Volume Setting Display. If the Prole LED and Volume setting are displayed, the Esteem® Query was successful. \*
5. If the corresponding LED is blinking, communication was not successful; repeat previous steps. \*

**Envoy Medical Corporation**  
4875 White Bear Parkway  
White Bear Lake, Minnesota 55110  
United States of America  
877-900-EARS (3277)  
[www.envoymedical.com](http://www.envoymedical.com)

Esteem and Envoy Medical are trademarks of Envoy Medical Corporation.

© 2020 Envoy Medical Corporation or its affiliates.

All rights reserved.

006203-001 Rev 03

**ENVOY** MEDICAL